

Mammoview Markers


Simple to use and compatible with any mammography x-ray system

These radio-opaque markers are designed to provide a clear indication of position, which can be read directly from x-ray film. They are manufactured in accordance with abbreviations recommended by the ACR.

Each marker is manufactured to be clearly visible on x-ray film only in the mammographic energy range.

Each set of markers comes with an acrylic holding device designed to be mounted near the x-ray unit for easy access.

The CIRS Mammoview Markers are quick and easy to use. Simply mount the acrylic holder near the mammography unit in close proximity to the bucky. Firm pressure applied to the suction cup will hold the marker in place on any smooth surface. Mammoview Markers are usable on any mammography system.

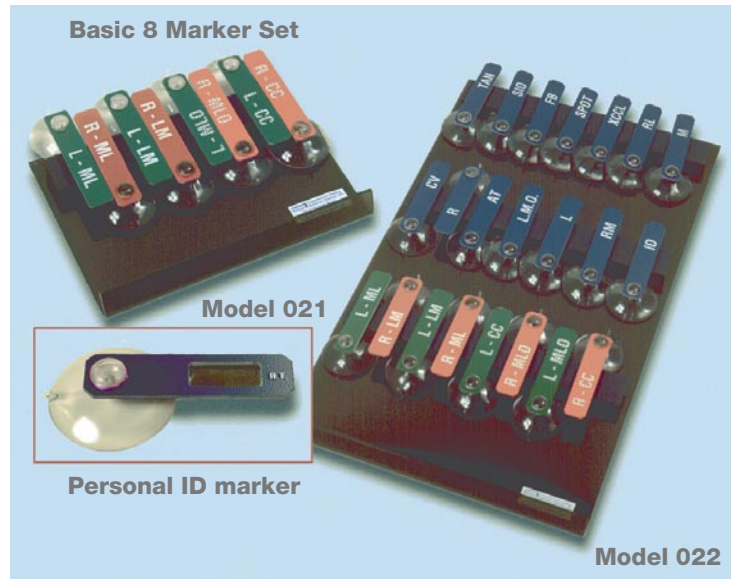

Features

- Fast and simple to use
- Meets all requirements for standardized terminology set forth by the American College of Radiology
- For all mammography systems
- Custom, personalized ID markers also available

*Tissue Simulation &
Phantom Technology*

CIRS

2428 Alameda Avenue • Suite 212 • Norfolk, Virginia 23513 • USA
(800) 617-1177 • (757) 855-2765 • FAX (757) 857-0523
www.cirsinc.com • admin@cirsinc.com

Marker Set Configurations:

Standard Set (8 Markers)

R-CC
R-MLO
R-ML
R-LM

L-CC
L-MLO
L-ML
L-LM

Full Service set (Standard Set +14 Additional Markers)

R
L
SPOT
M
CV
AT
ID
TAN
SIO
LMO
FB
RL
RM
XCCL

Personal ID markers are also available individually.

LATERALITY	LABELING CODE	PURPOSE
Right	R	
Left	L	
PROJECTION POSITION		
Mediolateral Oblique	MLO	Standard View
Craniocaudal	CC	Standard View
90° LATERAL		
Mediolateral	ML	Localize, Define
Lateromedial	LM	Localize, Define
Spot Compression	SPOT	Define
Magnification	M*	Define
Exaggerated Craniocaudal	XCCL	Localize
Cleavage	CV	Localize
Auxillary Tail	AT	Localize, Define
Tangential	TAN	Localize, Define
Rolled Lateral	RL*	Localize, Define
Rolled Medial	RM**	Define
Caudocranial (From Below)	FB	Define
Lateromedial Oblique	LMO	Define
Superolateral to Inferomedial Oblique	SIO	Define
Implant Displaced	ID	Augmented Breast

* Use a prefix before the projection. For example, RMMLO equals Right Magnification Mediolateral Oblique.

** Used as a suffix after the projection. For example, LCCRL equals Left Craniocaudal Upper Breast Tissue Rolled Laterally.

Excerpts taken from THE AMERICAN COLLEGE OF RADIOLOGY MAMMOGRAPHY QUALITY CONTROL MANUAL, 1994 Edition