

CIRS Motion Control – Upgrade Instructions

1. Download *CIRS Motion Control (last version#).zip* and unzip/uncompress it to a folder called *CIRS Motion Control (last version#)* in a known location.
2. Open the unzipped/uncompressed folder and check that it contains all the installation required files, respectively:
 - *MotionControlSetup.msi*
 - *Setup.exe*
 - *Install_TrioPCMotion_1_2_9_2.exe*

(**Note:** If *MotionControlSetup.msi* or/and *Setup.exe* /are missing please contact CIRS; if *Install_TrioPCMotion_1_2_9_2.exe* is missing please disregard its absence as this file is required for new installations only).
3. Go to *START – Control Panel – Add or Remove Programs* (Windows XP) or *Programs and Features* (Windows Vista).
4. Identify program called *Motion Control* and select *Remove* (Windows XP) or *Uninstall* (Windows Vista). Use the uninstall wizard to complete the process.
5. Unzip/uncompress *Motion Control 1.0.4.zip* to a folder called *Motion Control 1.0.4* in a known location. Open *Motion Control 1.0.4* folder.
6. Select and start *Setup.exe* file and follow the onscreen instructions to complete the installation of the new version.
7. Identify the *CIRS Motion Control shortcut* placed automatically on desktop and start the *CIRS Motion Control* software.
8. Go to *Help – About* and verify that the version of the software listed in the *About CIRS Motion Control* box matches the version of the software, which was last installed.

